


KN95 Protective Mask (Non Medical)

Way of wearing:

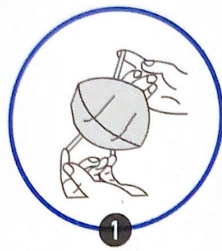

Warning Tips:

- When feeling breathing resistance is significantly increased, or when the mask is dirty or damaged, it should be replaced in time.
- This product should not be shared with others.

Please pay attention to the air tightness check after wearing the mask.



KN95 Protective Mask (Non Medical)

Product name: KN95 protective mask (Non Medical)

Type: disposable mask

Specifications: 10 PCS/ Box.

Filter efficiency: $\geq 95\%$.

Validity: 3 years from the production date.

Standard: GB2626-2006

Manufacturer: Zhangjiagang Sunrise Knitting and Apparel Co., Ltd.

Address: Xican Village, Fenghuang Town, Zhangjiagang City, Jiangsu Province, CHINA

Heqiao Village, Tangqiao Town, Zhangjiagang City, Jiangsu Province, CHINA

TEL: +86-512-82508457 82509988

Application:

- Suitable for adults' daily travel protection, not suitable for toxic gas environment use. This product cannot be replaced with gas masks and medical masks.
- This protective mask cannot generate oxygen and should not be used in hypoxic environments.
- It's better not to use the product if the air is not flowing, breathing is not smooth, or during sleep.
- Please use it correctly to avoid serious consequences caused by improper use.
- Store in a dry and ventilated place (temperature: $-5^{\circ}\text{C}\sim 50^{\circ}\text{C}$, relative humidity $< 80\%$).
- Masks are disposable and cannot be washed.

Please check for the air tightness after wearing the mask

Check the tightness of the mask to the face. Wearing the protective mask gently with both hands, please don't affect the position of the mask on the face, and then exhale quickly. If you feel that air leaks from the nose piece, re-adjust the nose clip according to step 4. If you feel air leaking from the edge of the mask, re-position the ear bands.

Warning Tips:

- When feeling breathing resistance is significantly increased, or when the mask is dirty or damaged, it should be replaced in time.
- This product should not be shared with others.

Production date: 05.2020

Batch no: 202005

MADE IN CHINA